Liberator

SETUP

KEY

SETUP/MODIFY

Technical Support:

01733 370 470 (Option 2)

Email: info@liberator.co.uk

Regional AAC Consultants:

KEY

ACCENT: Liberator Support



Click HERE to download NuVoice PASS

If you wish to speak to your Regional AAC

remotely connect to your device using an ID and

linked to a button in your vocabulary. This quick

Consultant, please call 01733 370 470 (Option 3)

Our Liberator Support module allows staff at Liberator to

PASSWORD. This file can be placed on your desktop and

reference auide will talk you through that process. Please

note this will only work if you are connected to Wi-Fi.

Toolbox

Tools

EDIT/SETUP



Support

Click HERE to download Liberator Support

Once you have downloaded this file place the file on your desktop.

1. How to start a Liberator Support Session

Select the **TOOLS** button Select **WIN/easyChat** or **Hide/Show NuVoice** You should now see the Windows Desktop Double tap on the Liberator Support Icon and y

Double tap on the Liberator Support Icon and wait 5 -10 seconds for Liberator Support to start.

Disclaimer		×
nformation. We will not sh Jberator for marketing pur https://www.liberator.co.u This Data is used only for p MPORTANT By starting the Jberator to support your d	retecting the privacy and security of your persential any your personal data with any company databits of privacy personal data with any without personal privacy. Alphanog participy presiding a technical support exercise, an partie sensity you are providing the GDPR consent for leaves enrolleding. Cohy atter this sension if you are happy to an up out of this investigation of you are happy to an up out of this investigation.	

Click **ACCEPT** to start Liberator Support

Now you should see Liberator Support displayed onscreen. The person who is supporting you will need to know: Your ID and Your PASSWORD



2. Creating a link that starts Liberator Support from vour vocabulary.

This instruction assumes you're adding it to the left offscreen buttons. To do this you must have the Liberator Support on your desktop. It may be in your downloads folder, if so copy and paste it to your desktop.

Select the TOOLS Select TOOLBOX Select VOCABULARY OPTIONS Select CHANGE OFF-SCREEN BUTTONS Choose a button to use (I have selected the one highlighted in the red box which typically has right mouse

click and isn't required)

Press CLEAR BUTTON CONTENTS Press INSERT TOOL Select **INSERT TOOL** Select SPELL TOOL TO FIND Type RUN Press OK Select RUN PROGRAM Select CHOOSE ANOTHER FOLDER Select USERS - ACCENT - DESKTOP Select "Liberator Support.exe" Select **INSERT TOOL** Select SPELL TOOL TO FIND Type **HIDE** and **OK** Select HIDE/SHOW NUVOICE Select OK Select CHANGE PICTURE Select SPELL ICON TO FIND Type the name of the picture you want Select OK Choose a picture Select CHANGE LABEL Type the name for the button label Select OK When finished, select **OK** to finish.

Support Contact: 01733 370 470 Option2